


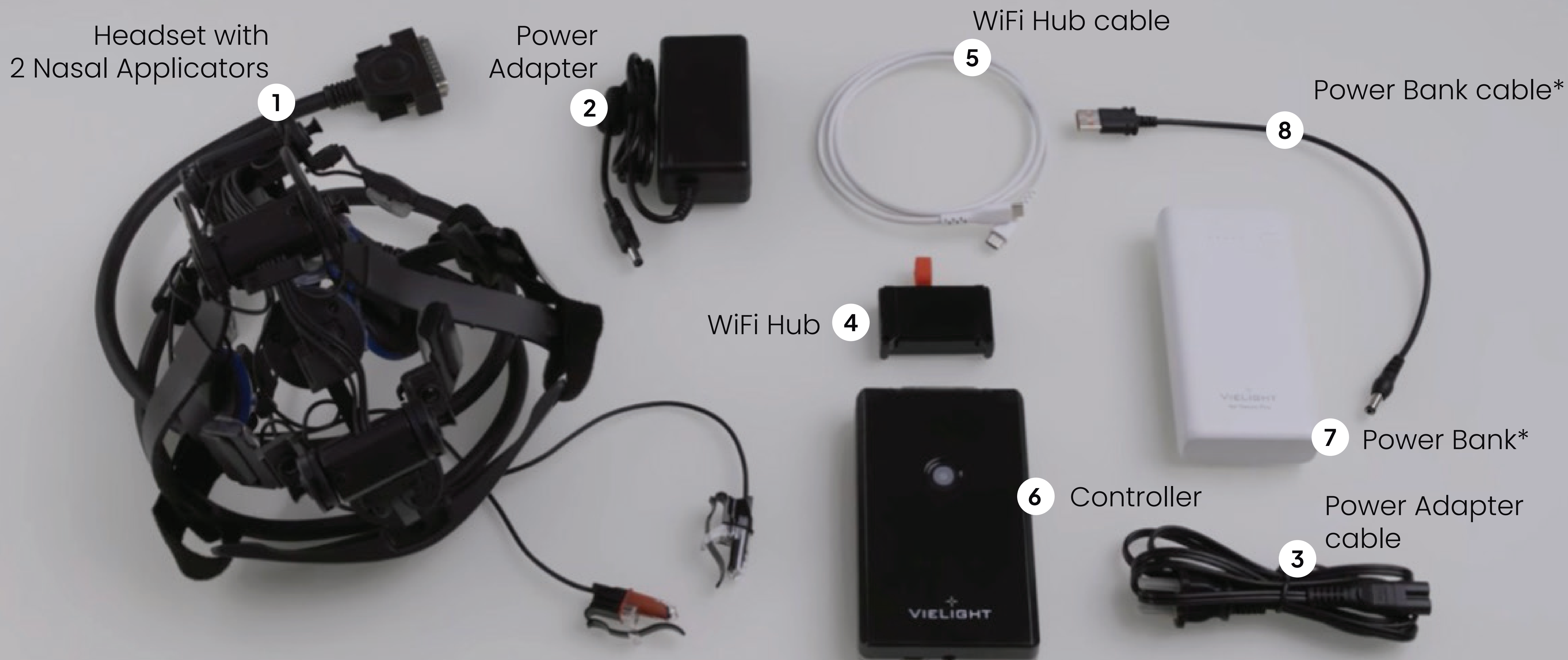

Powering up your Neuro Pro 2

1. Connect the power bank* 7 or power adapter 2 to the controller 6 using the power bank cable* 8 or power adapter cable 3
2. Attach the Wi-Fi hub 4 to the controller 6 using the Wi-Fi hub cable 5
3. Plug in the headset 1 to the controller 6


* Available only in the USA, Canada, and Australia. In all other regions, use the included power adapter instead of the Neuro Pro 2 power bank.

! Please do not use third party power banks with the Neuro Pro 2, it will destroy the circuitry.



Connecting your Neuro Pro 2 to your iOS Smartphone


2

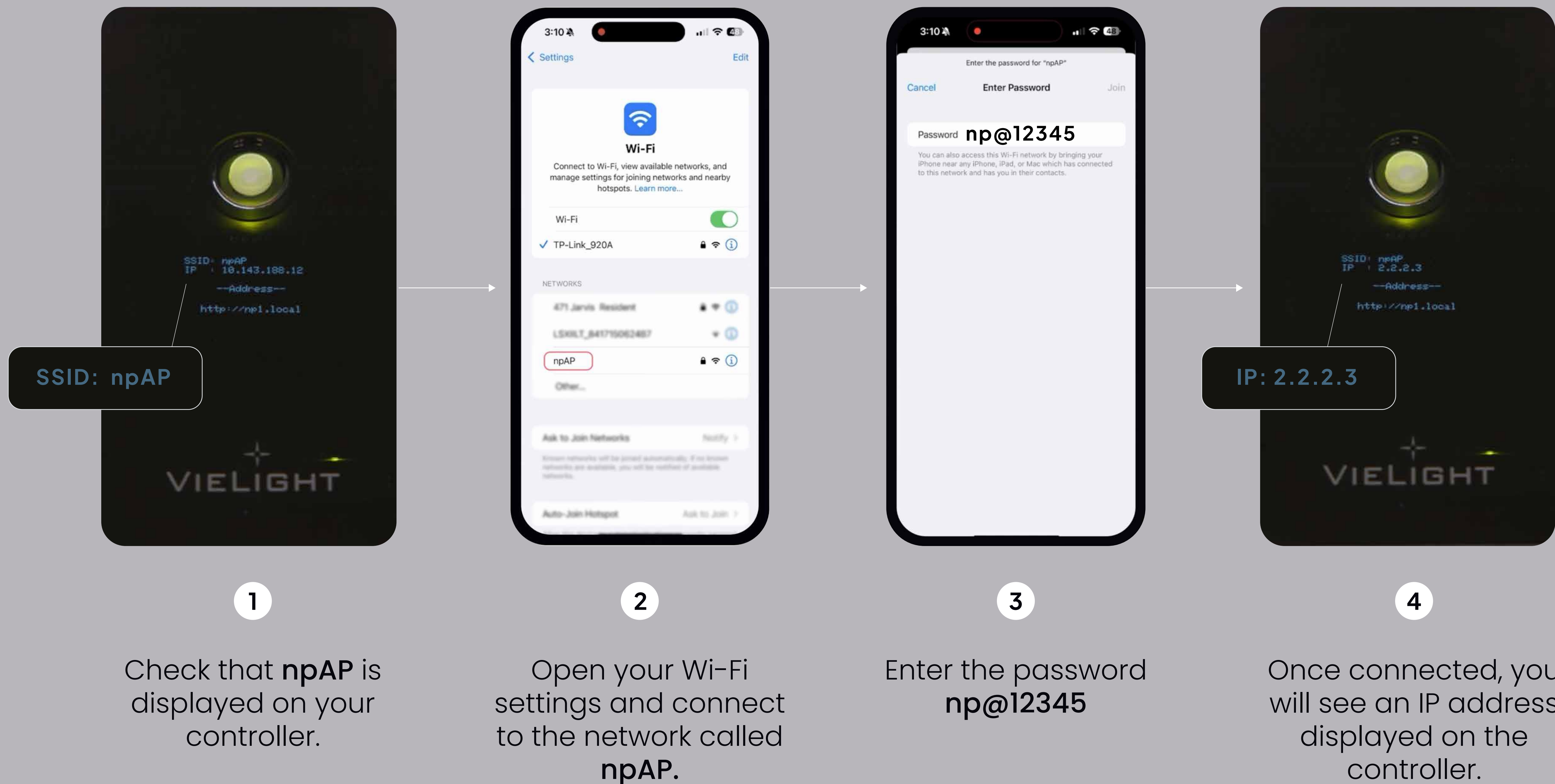




Connecting your Neuro Pro 2 to your iOS Smartphone

3

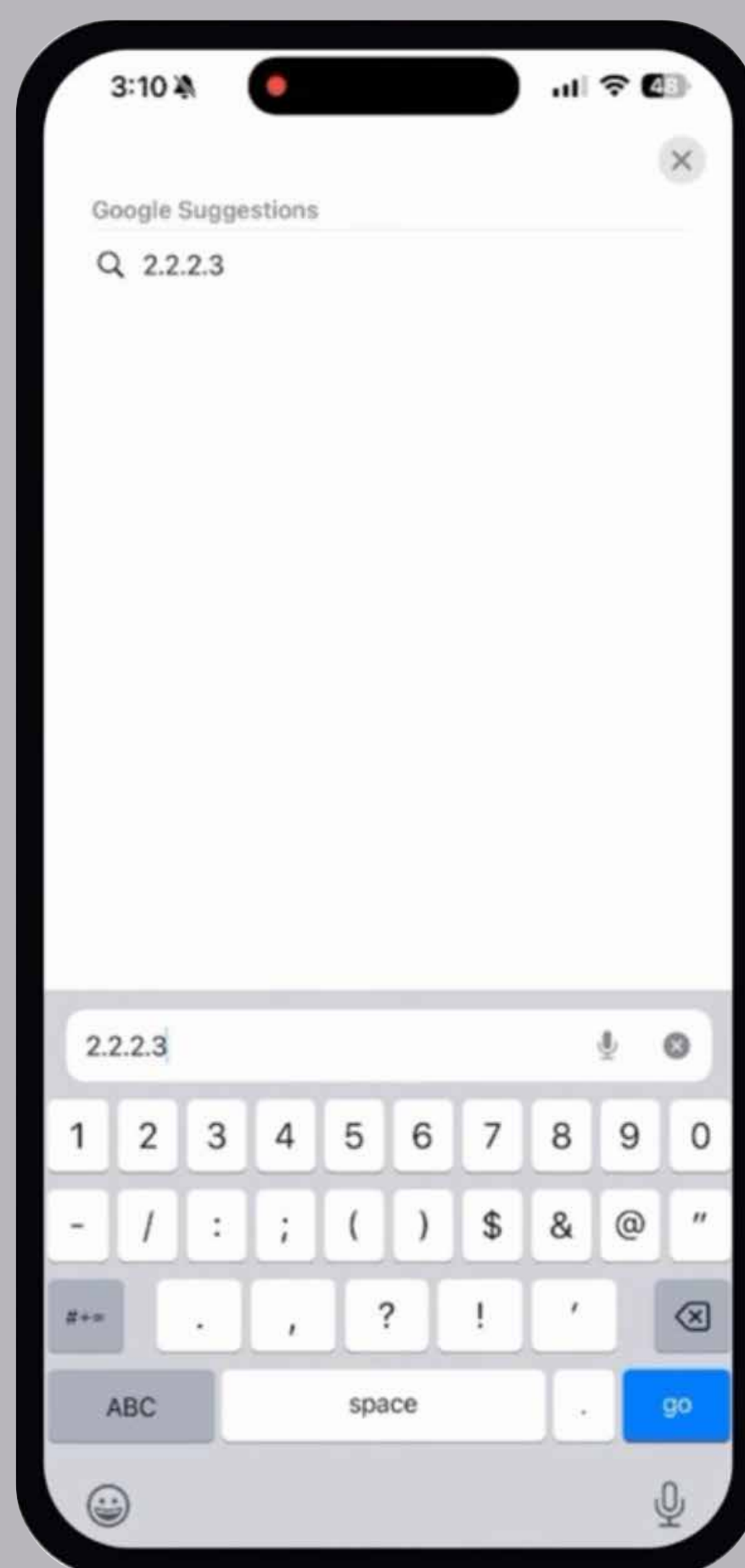

5

Open an internet browser on your iPhone and type in the IP address shown on the controller.

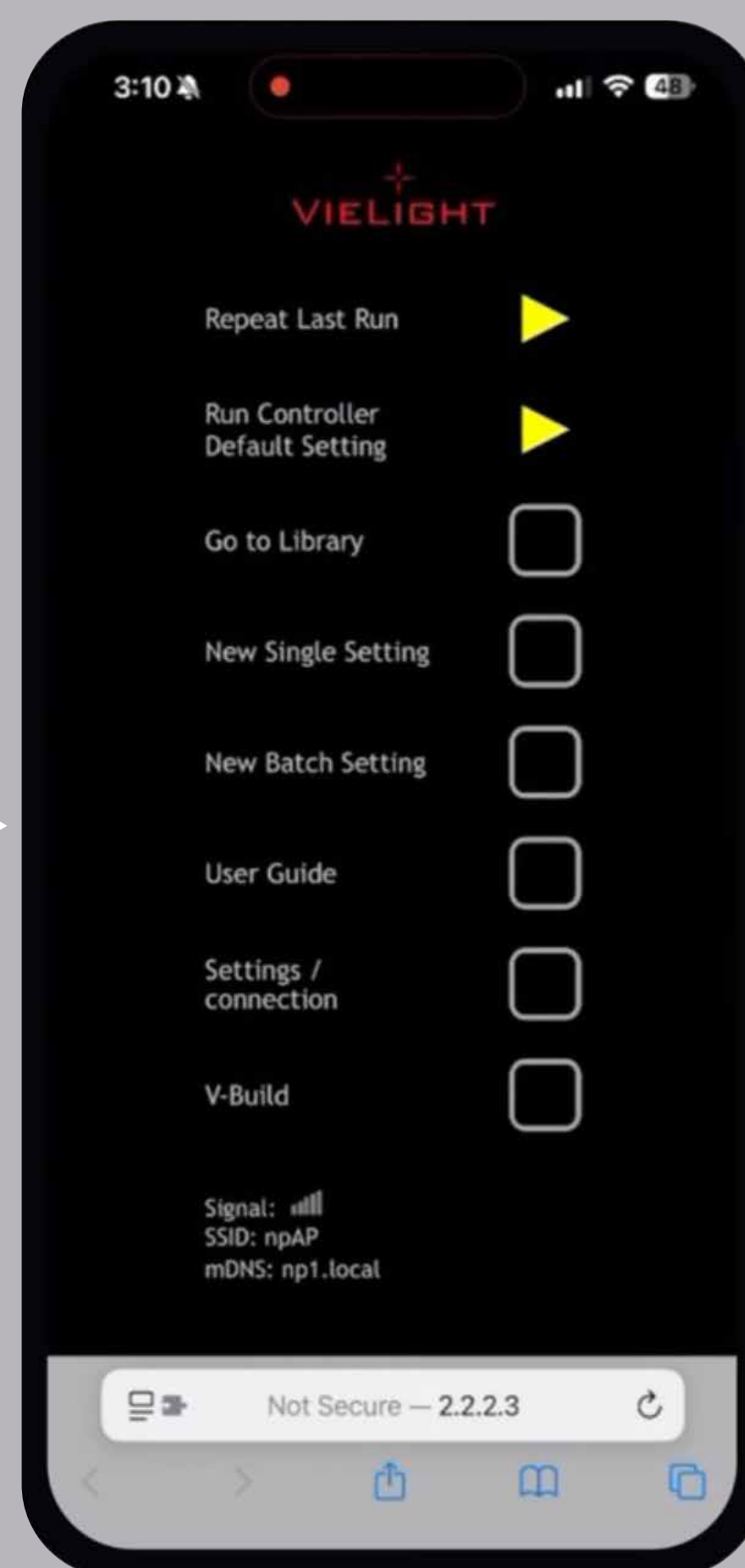

6

You are now connected to the Neuro Pro 2 GUI.



7


If you cannot connect, type in np1.local instead.



Connecting your Neuro Pro 2 to your iOS Smartphone


4

Optional and recommended: Connecting the Neuro Pro 2 to your Personal Wi-Fi network (enables an Internet connection while using the Neuro Pro 2)

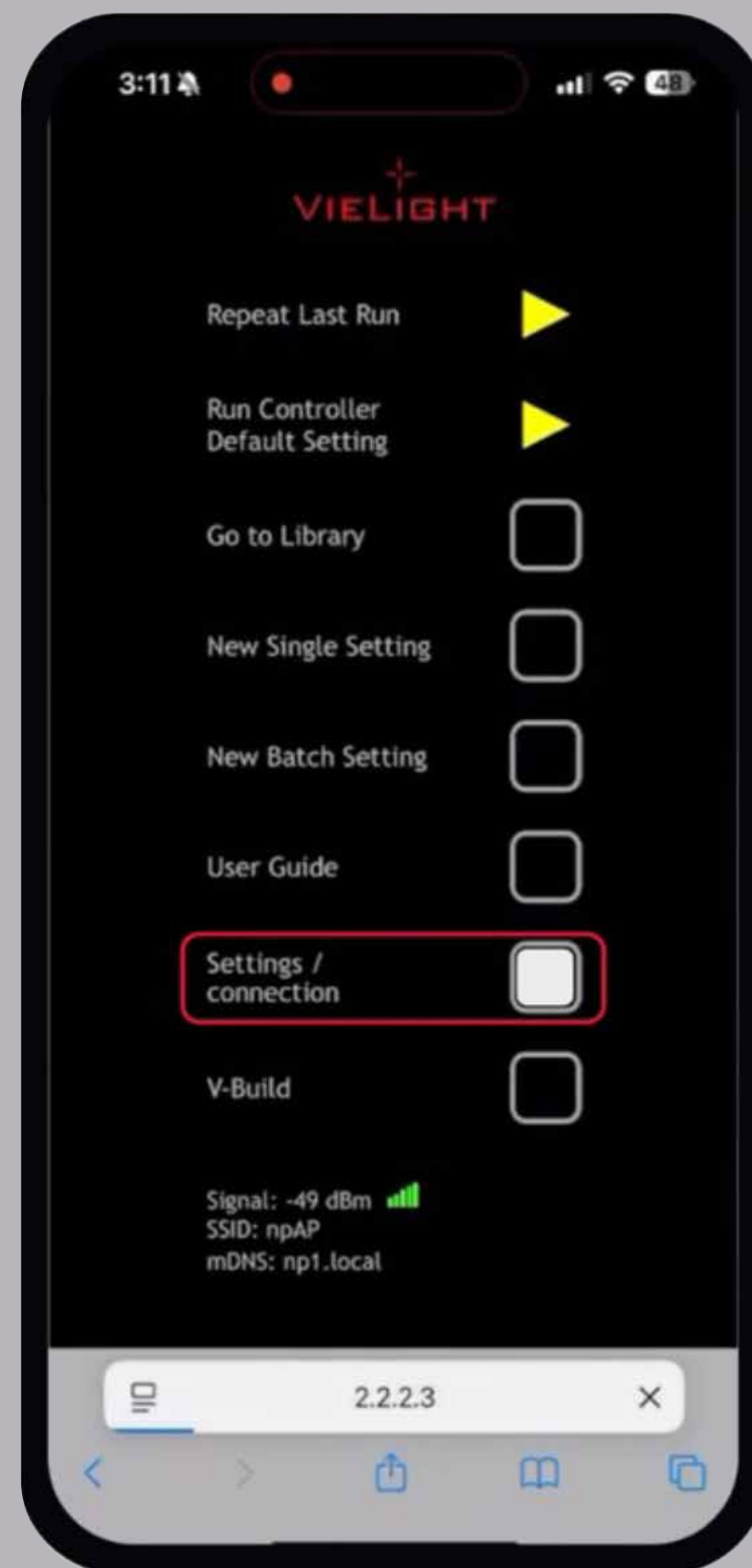


1

In the **GUI**, go to Settings / Connection

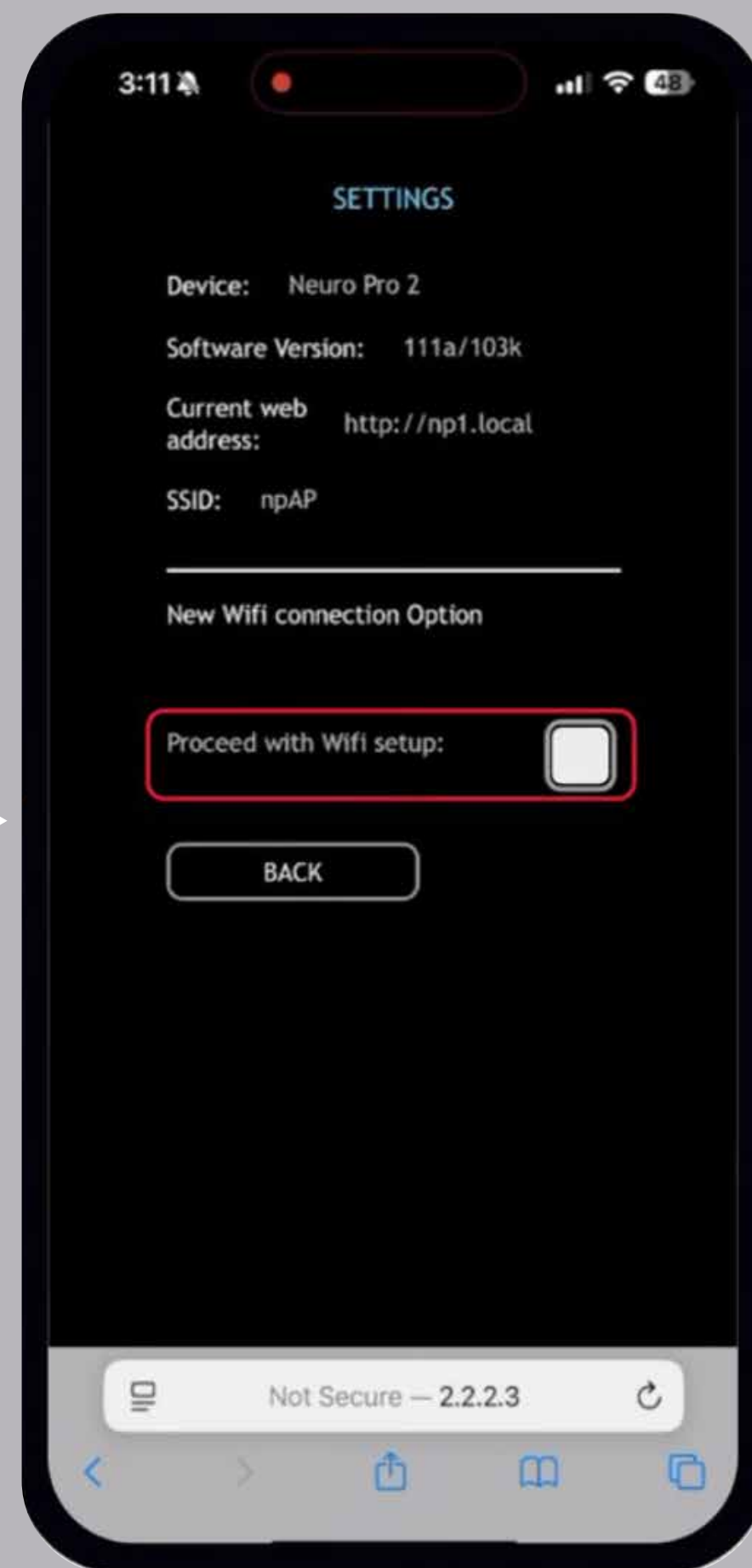


2

Press **Proceed with Wi-Fi Setup**.

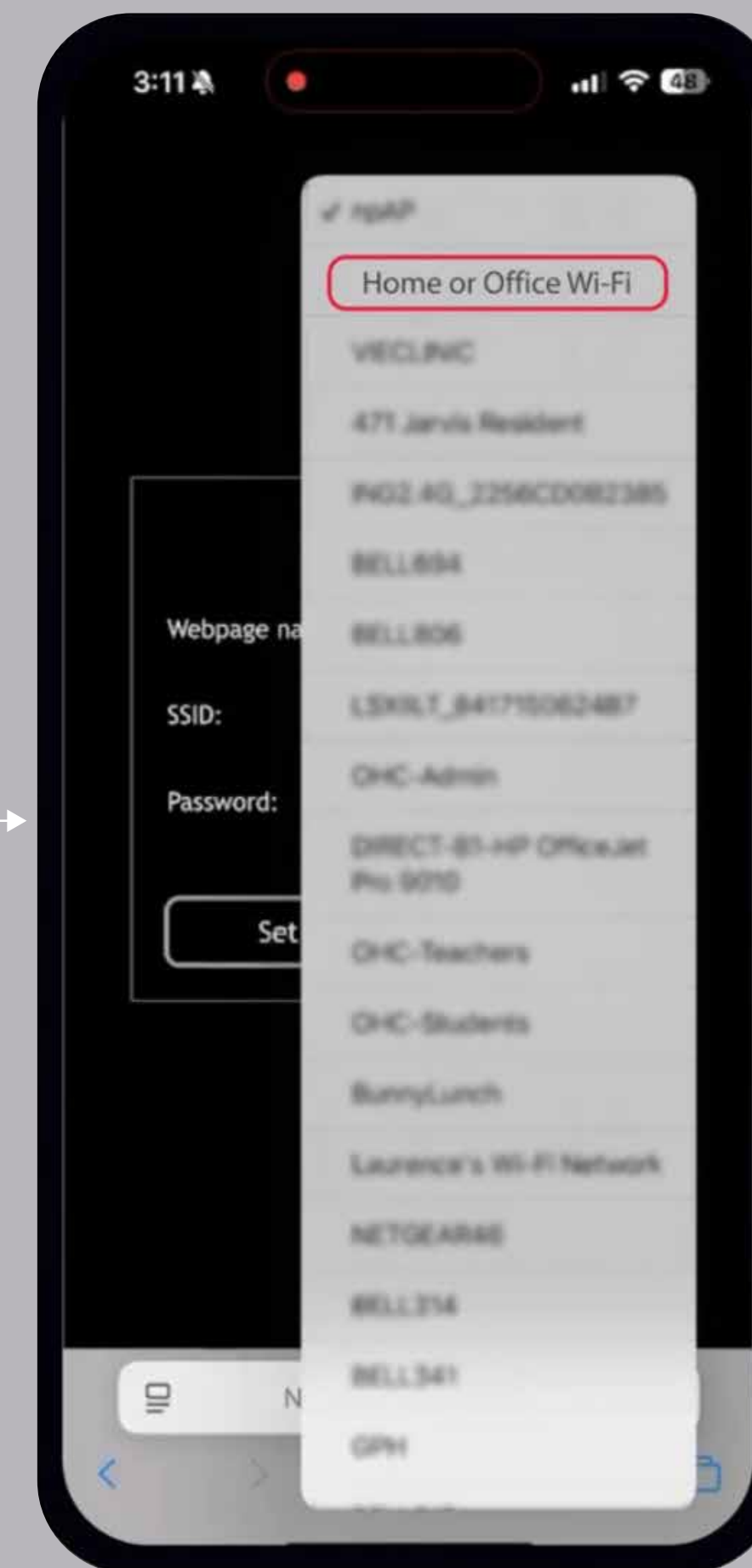

3

The Neuro Pro 2 GUI will scan for available Wi-Fi networks. Select your home or office network and enter the password.

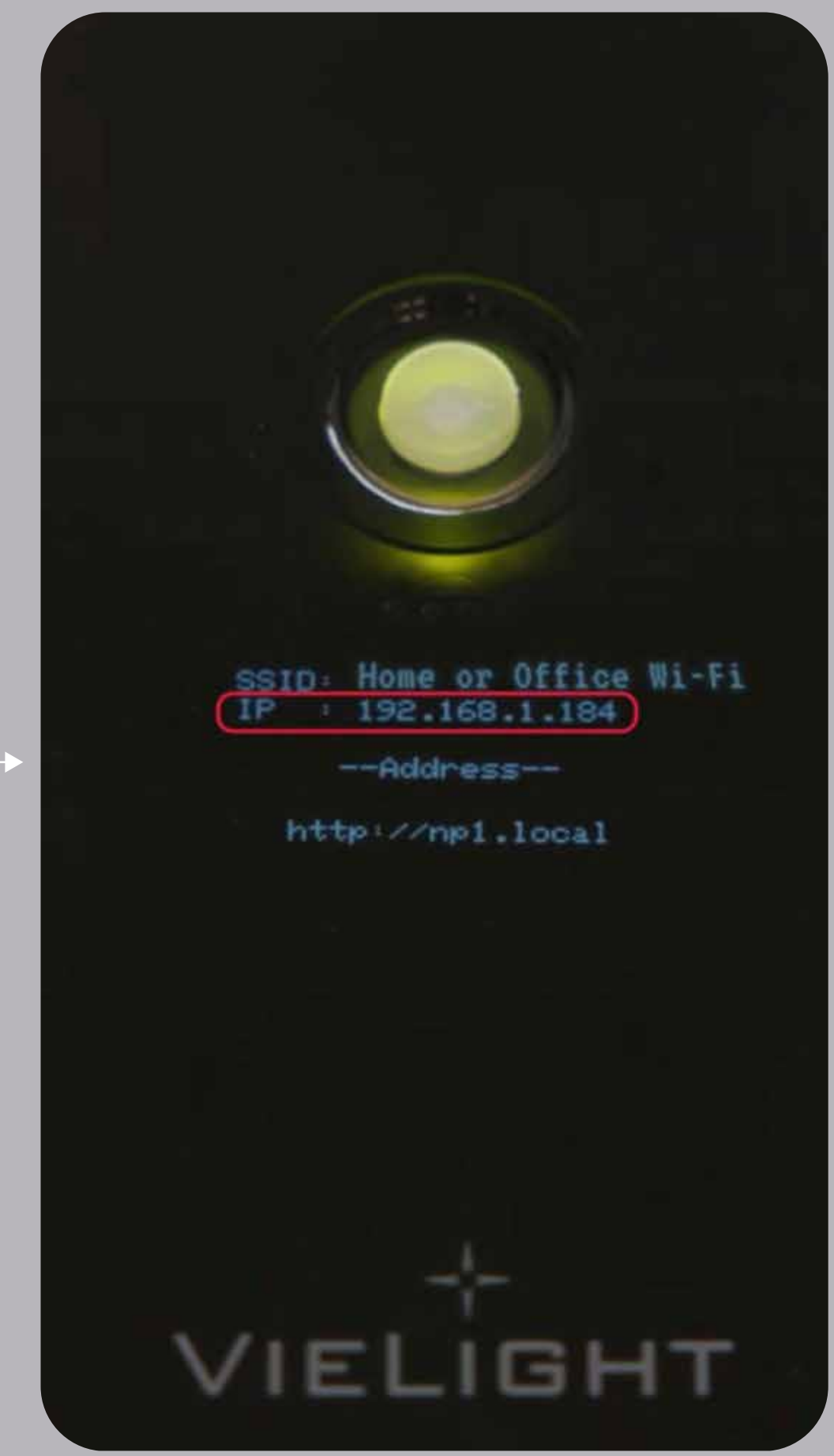

4

You will then see the controller IP change from offline to a new **IP address**.




Connecting your Neuro Pro 2 to your iOS Smartphone

5




5

On your iPhone, connect to **the same Wi-Fi network** you just linked the **Neuro Pro 2** to.

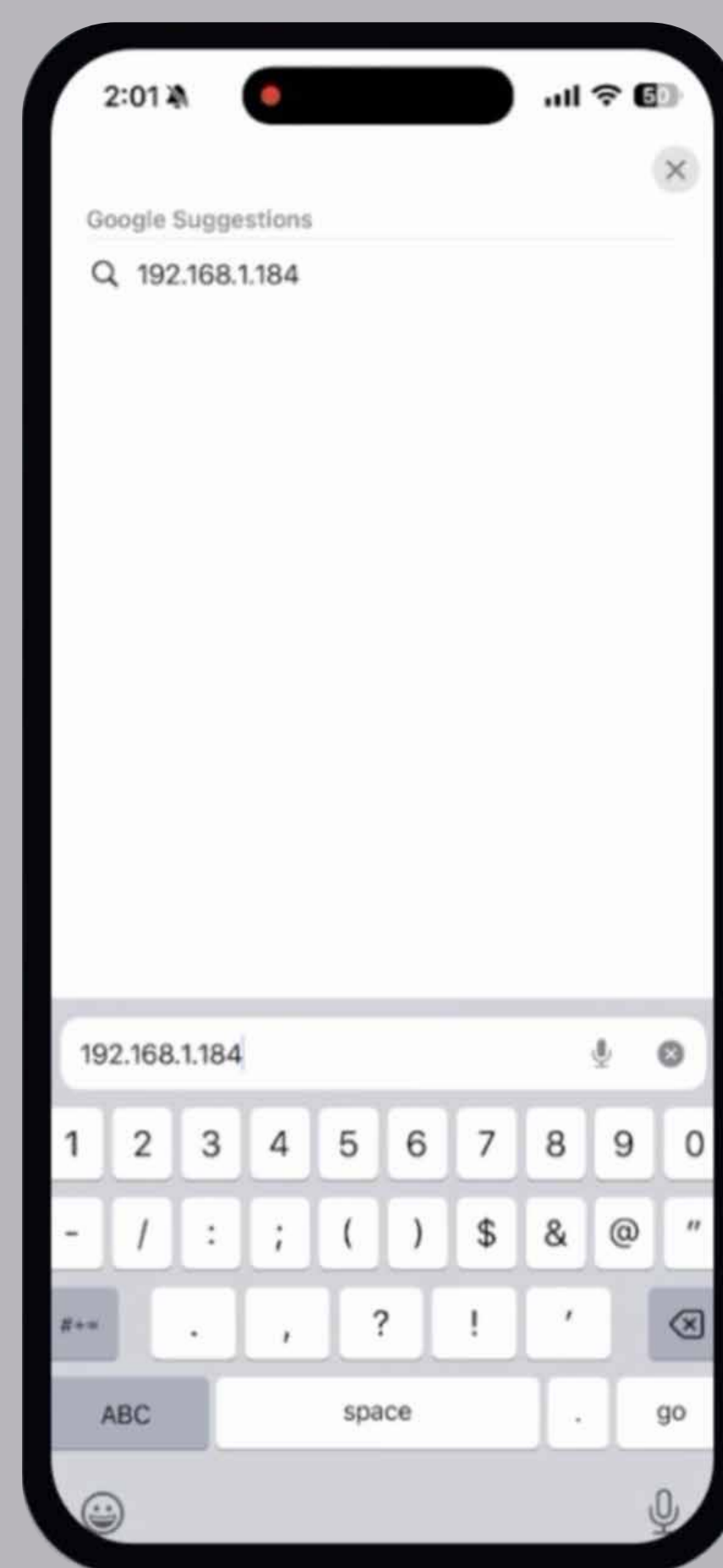

6

Open an internet browser and type in the **IP address** displayed on the controller.

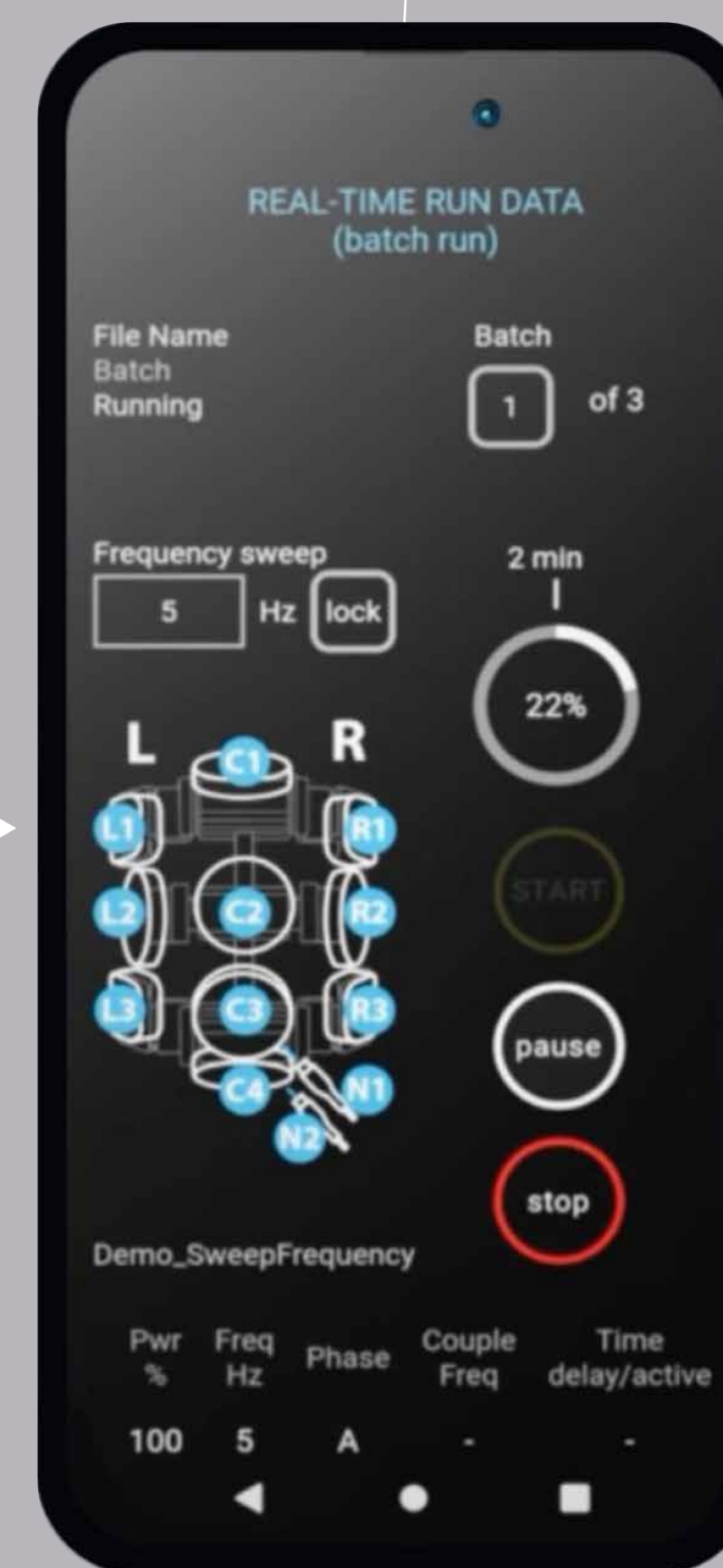

7

That's it! You are now connected to **the Neuro Pro 2 GUI** and can proceed with the settings and control of your device.

Please **ensure** that your iPhone screen shows the **Neuro Pro 2** interface. Keep your phone screen ON for the entire duration of the session and remain on the same page without navigating away.

Any screen timeout or page switching will interrupt the session run. The **Neuro Pro 2** was designed for continuous communication with your personal device — future-proofed for our **Neuro AI platform**.




Optional Feature

6


If there is a program you often use, you can set it to run automatically whenever you press the **Neuro Pro 2** power button, without needing your smartphone.

To do so:

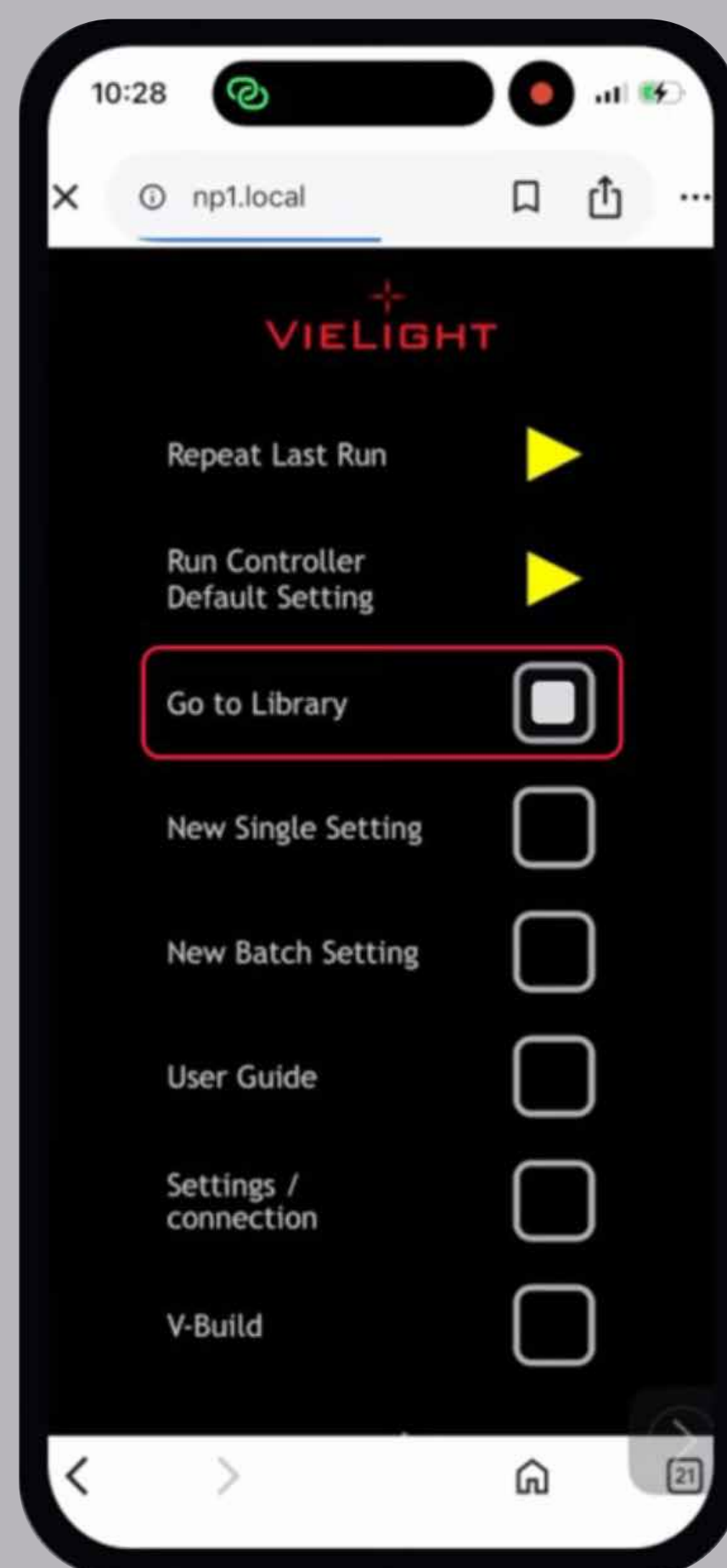

1

In the **GUI**, select **Go to Library**.

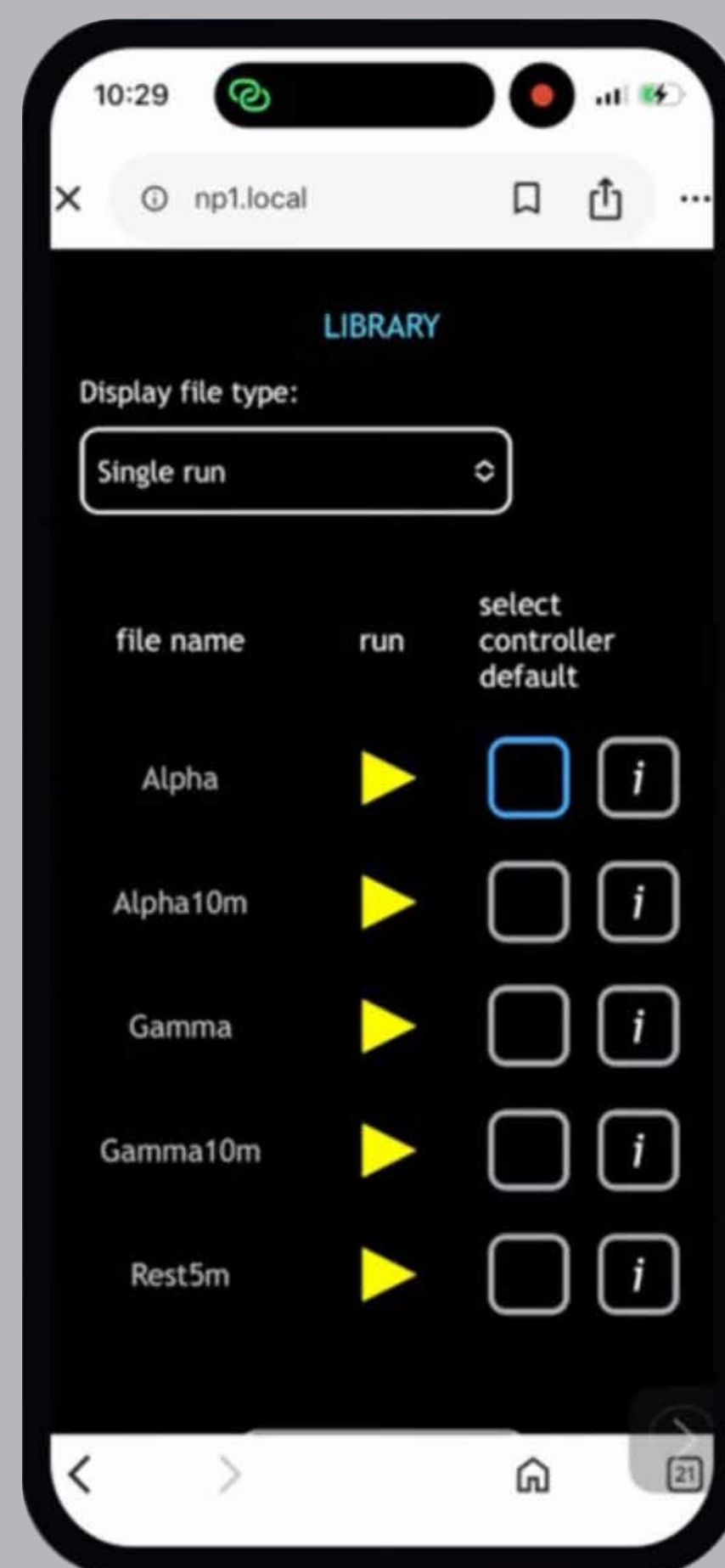

2

Make sure **Display** file type is set to **Single Run**.*



3

Press the **square icon** in the select controller default column next to your chosen program.

Now, you can start that program simply by pressing the power button after turning on your **Neuro Pro 2** – no connection required.


!

* Please note that only single-run protocols can be set as the controller's default. Batch files cannot be assigned as defaults.

! How do I connect my personal device to the Neuro Pro 2 controller?


7

Follow the steps below:

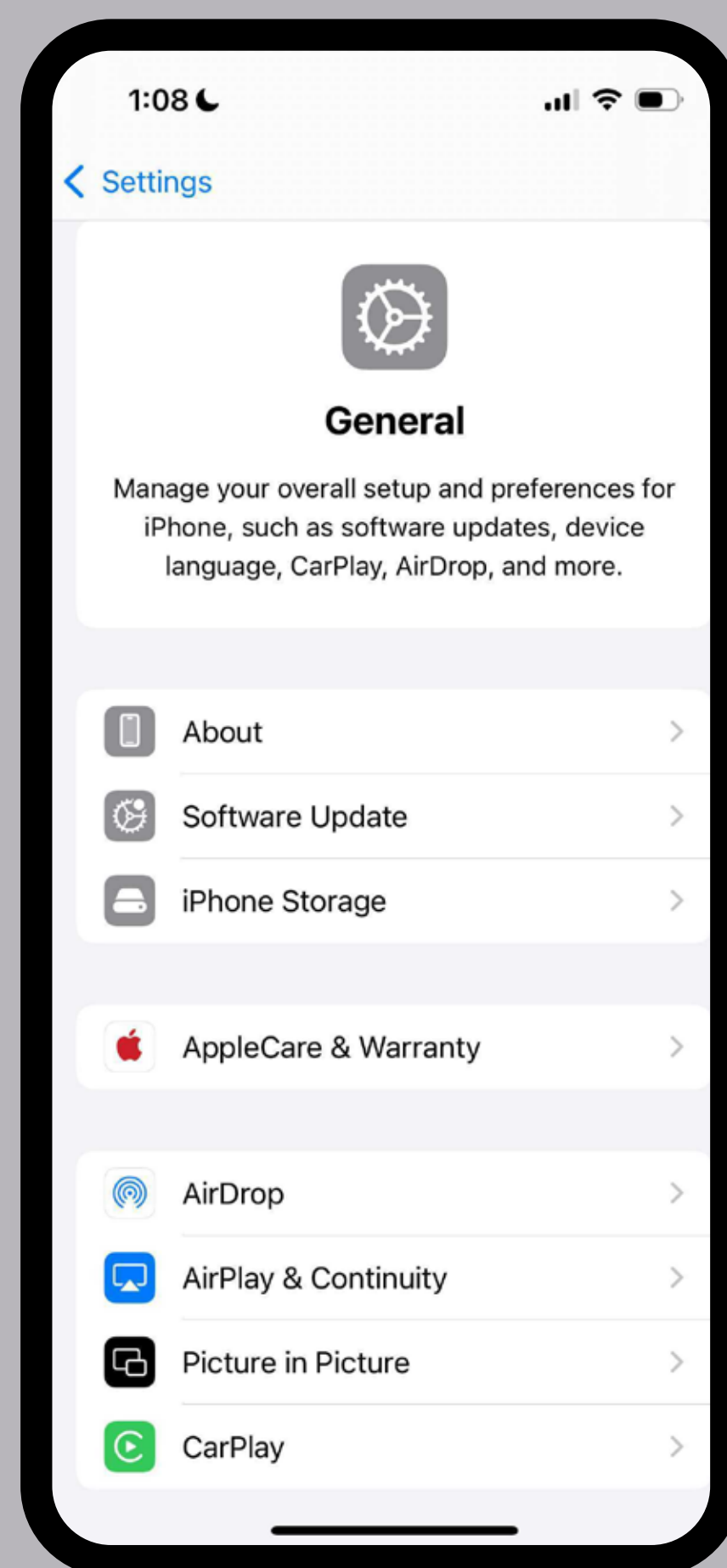


1

Go to Settings -> General -> About.

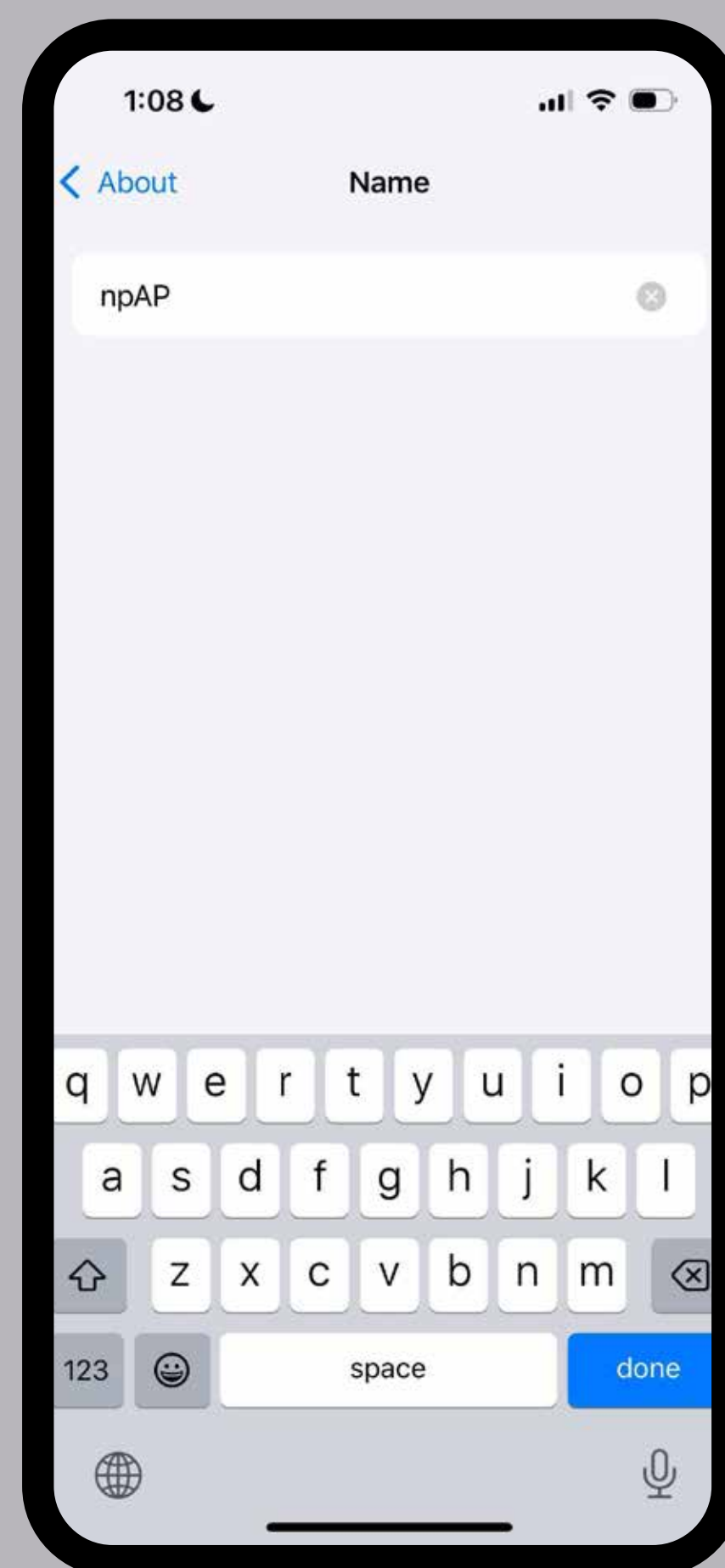

2

Change your phone's name to "npAP."

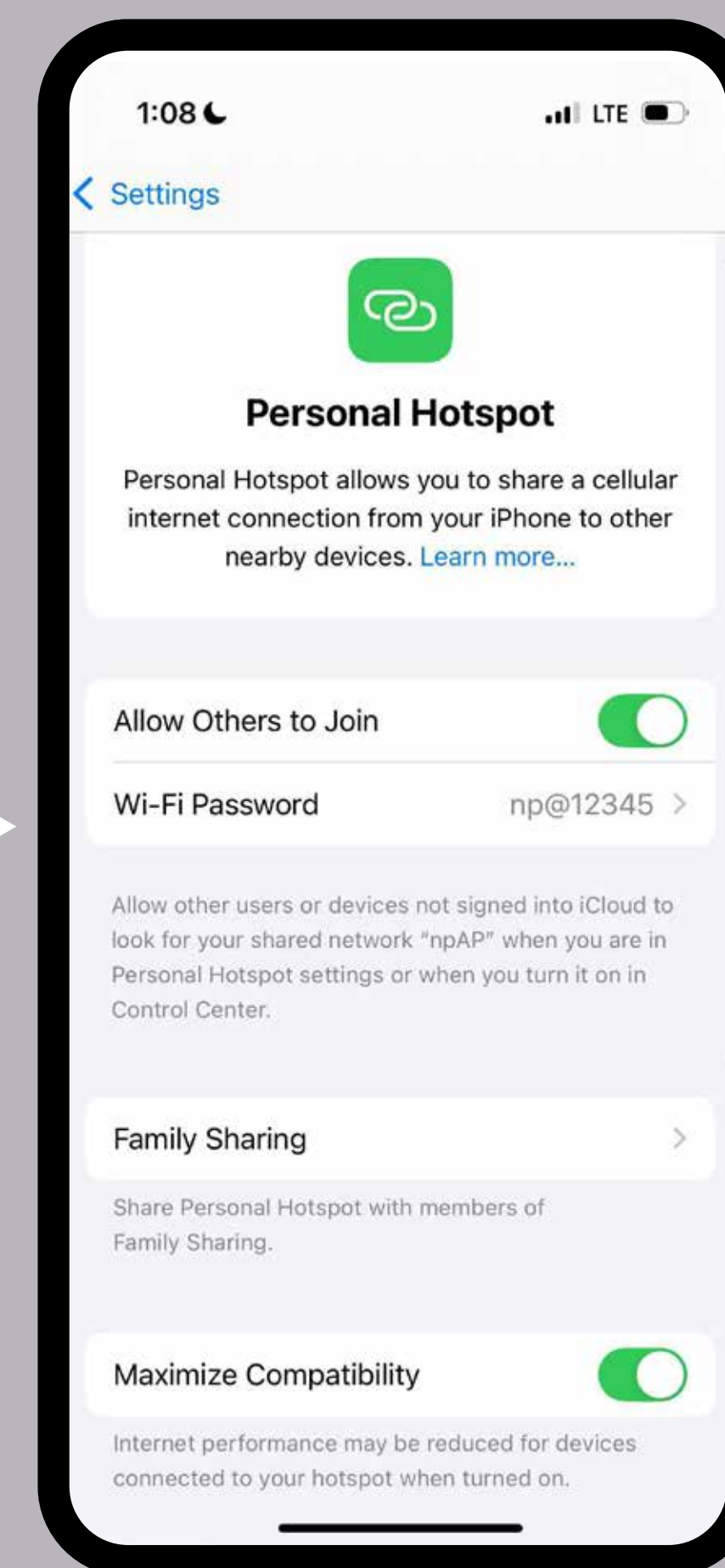

3

In Settings -> Personal Hotspot, enable "Allow Others to Join," set the password to "np@12345," and turn on "Maximize Compatibility."



4

Open a browser and go to **http://np1.local** to access the **GUI**.